

# Quantum Imaging and Therapeutic Associates

## Abdomen/Pelvis (Trauma) – Toshiba Aquilion One / 64

Indications	Trauma
Position / Landmark	Supine / Xiphoid
Scan Type	Helical
Start / End Locations	1 cm above diaphragm / lesser trochanters
FOV	38 cm (or as small as feasible)
kV / mA	120 kV / Sure Exposure
Slice Thickness/Interval	5 mm x 5 mm
IV Contrast Volume / Rate	75 cc Omni 350 @ 2 cc/s
Scan Delay / Sure Start	65 seconds
Oral Contrast	None
MPR	Coronal and sagittal 5 mm x 5 mm
Images to PACS	Scouts, axial soft tissue, MPR
Comments / Special	N/A